



Quick-Stop silver nitrate

For animal use

Product description

Veterinary pencils – Product ref: 81QS

- Silver nitrate 95% with 5% potassium nitrate
- Plastic holder with cap
- Single animal, multiple use product

Veterinary applicators – Product ref: 453QS

- Silver nitrate 75% with 25% potassium nitrate
- Bendable plastic handle
- Single-use product

Uses

Used for superficial skin lesions and minor wounds to stem bleeding, including the use on claws.

Administration

Simply moisten the tip using a small drop of water and apply to affected area to allow for cauterisation.

The silver nitrate pencil tip is fragile. To avoid breakages do not apply pressure during treatment and dry the pencil carefully after use, by dabbing on a paper towel or tissue and replace cap.

Disposal

Follow local waste regulations.

Contraindications and warnings

Take care when using the product as silver nitrate can stain black*. Staining of the skin will shed after several days.

*Avoid contact with work surfaces and healthy skin.

Do not use near eyes.

Precautions

Store in original package to protect from light and moisture.

Keep out of sight and reach of children. Not for use near the eyes and or other sensitive areas of the animal.

Poisoning by oral ingestion is unlikely with the quantities involved, but treatment for poisoning should be with repeated stomach washouts with 1% sodium chloride solution, followed by a solution of 30 grammes of sodium sulphate in 250ml of water, to be allowed to remain in the stomach. Please speak to a healthcare professional if you are concerned.