

Quick-Stop by Bray silver nitrate

For animal use

Product description

Quick-Stop by Bray silver nitrate pencil 95%

Product ref: 81QS

- Silver nitrate 95% and potassium nitrate 5%
- Plastic holder with cap
- Single-animal, multi-use product

Quick-Stop by Bray silver nitrate applicators 75%

Product ref: 453QS (50) Regular

Product ref: 454QS/10L (10) Large

- Silver nitrate 75% and potassium nitrate 25%
- Flexible plastic handle with treated tips
- Single-use product

Uses

Used for superficial tissue damage such as minor wounds, cuts, scrapes, and scratches, including after grooming and nail clipping.

Caution

- Silver nitrate naturally stains and will mark healthy skin, clothing, and hard surfaces black. Protect these areas before use.
- Skin staining will appear several hours after application and will fade over a few days.
- Staining to clothing and hard surfaces could be permanent. For cleaning tips, search online "silver nitrate staining".

Warnings and precautions

- Store in the original package to protect from light and moisture.
- Keep out of the sight and reach of children.
- Avoid contact with healthy skin and eyes.
- Not for use near eyes or other sensitive areas of the animal.
- Poisoning by oral ingestion is unlikely with the quantities involved. In case of ingestion, treatment should include repeated stomach washouts with 1% sodium chloride solution, followed by a solution of 30 grams of sodium sulphate in 250 ml of water to remain in the stomach. Consult a healthcare professional if concerned.

Directions for use

Quick-Stop by Bray silver nitrate pencil & applicator

Intended use: For topical application to the affected area only

1. Moisten the tip

Lightly moisten the tip with a drop of water or allow natural moisture from the area to activate the silver nitrate. Transfer the solution from the tip (either pencil or applicator) to the affected area, no pressure is required.

2. Apply to affected area

Apply gently to the affected area only. Once dissolved onto the minor wound, silver nitrate begins to cleanse and soothe the tissue. The silver nitrate pencil tip is fragile—do not apply pressure. Avoid contact with healthy skin.

3. After each use

- **Pencil (multi-use):** Gently dab the tip with tissue to remove excess moisture and replace the cap to preserve the tip from breakage.
- **Applicator (single-use):** Dispose of the applicator after use in accordance with local regulations.


4. Protect from staining

- Silver nitrate may stain skin and surfaces black. Protect surrounding areas during use.
- Skin staining is temporary and will naturally shed after several days.

1. Moisten the tip



2. Apply to affected area

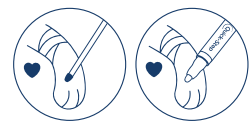

3. After each use



4. Protect from staining



Disposal

Follow local waste regulations for disposal.