

## Quick-Stop by Bray silver nitrate

### For animal use

#### Product description

##### Quick-Stop by Bray silver nitrate applicators 75%

Product ref: 453QS (50) Regular

Product ref: 454QS/10L (10) Large

- Silver nitrate 75% and potassium nitrate 25%
- Flexible plastic handle with treated tips
- Single-use product

#### Uses

Naturally supporting minor wounds and superficial tissue damage. Use after clipping nails and for minor scrapes and scratches.

#### Caution

- Silver nitrate naturally stains and will mark healthy skin, clothing, and hard surfaces black. Protect these areas before use.
- Skin staining will appear several hours after application and will fade over a few days.
- Staining to clothing and hard surfaces could be permanent. For cleaning tips, search online “silver nitrate staining”.

#### Warnings and precautions

- Store in the original package to protect from light and moisture.
- Keep out of the sight and reach of children.
- Avoid contact with healthy skin and eyes.
- Not for use near eyes or other sensitive areas of the animal.
- Poisoning by oral ingestion is unlikely with the quantities involved. In case of ingestion, treatment should include repeated stomach washouts with 1% sodium chloride solution, followed by a solution of 30 grams of sodium sulphate in 250 ml of water to remain in the stomach. Consult a healthcare professional if concerned.

#### Directions for use



##### 1. Moisten the tip

Moisten the tip with a drop of water and carefully transfer the solution from the tip to the affected area. Alternatively, moisten the tip using the liquid from the affected area.



##### 2. Apply to affected area

Apply to the affected area only, no need to apply pressure. Applicators are for single use, once used dispose of the applicator.



##### 3. Protect from staining

Silver nitrate can stain black. Protect all surface areas and avoid contact with healthy skin and eyes. Staining of the skin will shed after several days.

#### For Quick-Stop by Bray silver nitrate pencil 95%

see Instruction for use leaflet on [www.bray-healthcare.com](http://www.bray-healthcare.com)

#### Disposal

Follow local waste regulations for disposal.